

2net™ Hub by Qualcomm Life

QUICK START GUIDE

QUICK START GUIDE Easy 1-2-3

1


Plug in your 2net™ Hub to start the self-running activation process. The bar and home icons light up to indicate activity. This may take up to 5 minutes to complete.

SOLID YELLOWS



Plug in and Start Up
Automatic Activation

FLASHING YELLOWS

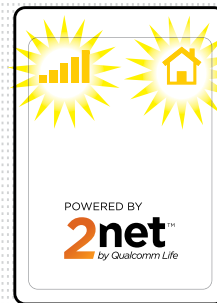

Initializing

SOLID GREEN/YELLOW



Ready for Pairing
with your Medical Device



2

The 2net Hub pairs and collects medical data from your device.

FLASHING GREEN HOME



Data Transfer from
Device in Progress
(Begins Automatically)

*Note: For USB
capable devices, first
connect the 2net
Hub to your medical
device using a
USB cable*

3

The 2net Hub sends collected data over a cellular network.

FLASHING GREEN BAR



Wireless Network
Activity:
Transmitting Data

FLASHING BLUE LIGHT



Successful
Data
Transmission

**SOLID GREEN/
YELLOW**



Ready for
Pairing with your
Medical Device



80-KA132-2 Rev D