English العربية للغة Болгарски 简体中文 繁體中文 Hrvatski Čeština Dansk Nederlands Eesti
Suomi Français Deutsch Ελληνικά עברית Magyar Italiano 日本語 한국어 Latviski Lietuvių k.
Malti Norsk Polski Português Română Slovenčina Slovenščina Español Svenska

Philips Axon Safety Instructions

PHILIPS

Contents

English.....	2
اللغة العربية	8
Български.....	11
简体中文	16
繁體中文	19
Hrvatski.....	22
Čeština.....	27
Dansk.....	32
Nederlands.....	38
Eesti.....	43
Suomi.....	47
Français	52
Deutsch	58
Ελληνικά.....	63
עברית	68
Magyar.....	71
Italiano.....	76
日本語	81
한국어	84
Latviski	87
Lietuvių k.....	91
Malti.....	95
Norsk	100
Polski.....	105
Português.....	110
Română.....	115
Slovenčina	120
Slovenščina	125
Español.....	130
Svenska	135
Philips	140

English

IMPORTANT SAFETY INFORMATION

Before using the Philips Axon, make sure to read all the safety information provided below, as well as the *Philips Axon Instructions for Use* available at customers.capsuletech.com.

Scan the QR code below to access the customer portal and find the relevant product documentation using your product's label.

- Be careful to cut only the tape and not, for example, the Philips Axon.
- Fire and explosion hazard. Do not operate the Philips Axon in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide; in oxygen-enriched environments; or in any other potentially explosive environment.
- Do not place the Axon inside furniture or inside any enclosed space. This could prevent correct cooling of the device and interfere with radio communications.
- Operate the Axon only within the environmental conditions described in the “Technical characteristics” of the *Philips Axon Instructions for Use*.
- Do not stack the Axon or place it next to other equipment. If you do, then check it for normal operation running its usual configuration.
- For best practice, mount the Axon on a wall or place it on a tabletop away from high traffic areas. Keep cables out of reach and off the floor.
- Secure overhead installations of Philips Axon to prevent falls. Perform regular maintenance checks to ensure stability.
- Do not use extension cords or adapters of any type.
- Do not use a power supply unit that is not the one provided by Philips (listed in the *Philips Axon Instructions for Use* document). The use of another power supply unit can result in a short circuit.
- The power supply unit must remain accessible to unplug the device from the power supply mains if necessary.
- Do not connect a PoE injector to the serial port to avoid damaging the device.

English

- When you connect passive PoE to a serial port, you risk damaging the Axon. Read the port labels carefully to avoid making this mistake.
- Do not make any modifications to this equipment.
- If the device enclosure is broken, do not touch the Axon, and stop using the device.
- Electric shock hazard. If a power surge is propagated through the wired network connection at the time of the connection setup procedures described in this section, there is a residual risk of electric shock if you touch the metal accessible parts of the DIM and cables. Either:
 - Disconnect the Ethernet cable before you connect the DIM to the Axon.

Or

- Connect the DB9 cable to the DIM and to the medical device, then connect the RJ45 cable to the DIM, then connect the RJ45 cable to the Axon.
- If you use unapproved cables to connect medical devices to the Axon,

you could cause damage to or malfunctioning of the medical device.

- For safety reasons, cables are designed to prevent inadvertent disconnection if someone accidentally pulls on them. Do not route cables such that they present a tripping hazard. For devices that are installed above the patient, take adequate precautions to prevent them from falling on the patient.
- Electric shock hazard. Do not open the Axon or attempt repairs. The Axon has no user-serviceable internal parts. Inspection and servicing of internal parts shall only be performed by qualified service personnel.
- To avoid damage to the Axon while cleaning it, see the "Periodic cleaning and disinfection" section in the *Philips Axon Instructions for Use*.
- Since the Axon is not completely watertight:
 - Do not spray liquid or foam on the Axon or submerge or soak it for cleaning.
 - Do not spray or squirt any type of liquid directly on the Axon.

English

Instead, spray the liquid, gel, or foam on a cloth and then use that cloth to rub on the device.

- Do not spill liquids on or around the unit.
- Do not expose the power cable and power supply unit to liquid.
- Do not use any other agent than those listed in the *Philips Axon Instructions for Use*. (Contact Philips Capsule for assistance if your cleaning agent is not on the list.)
- Do not use harsh solvents.
- Do not clean, disinfect, or sterilize any part of the system by autoclaving, or with the use of steam, or heat, or ethylene oxide gas (gas sterilization).

Electrical Specifications

Power Supply: 100–240 V AC,
50/60 Hz / Power-over-Ethernet: 36 V – 57 V, up to 15W

Frequency Bands and Power of the Philips Axon

Wi-Fi 2.4 GHz band: 2412 MHz to 2472 MHz
(Power < 19.9dBm)

Wi-Fi 5.0 GHz band: 5180 MHz to 5240 MHz
, 5260 MHz to 5320 MHz, 5500 MHz to

5700 MHz, 5725 MHz to 5850 MHz (Power < 22.99 dBm)

Operating environment

Temperature: 10°C to +40°C

(+32°F to +104°F)

Humidity: 10% to 90% RH non-condensing

Atmospheric pressure:

800 hPa to 1060 hPa

Transport and storage environment

Temperature: -20°C to +60°C

(-4°F to +140°F)

Humidity: 10% to 90% RH non-condensing

Atmospheric pressure:

500 hPa to 1060 hPa

Important note

Any changes or modifications to this product are strictly forbidden. Such actions will immediately void all safety and wireless communication certifications and revoke your authorization to operate the device. This product has been evaluated and certified to meet safety and wireless standards when used with approved accessories and shielded cables that comply with regulatory requirements. To prevent interference with radios, televisions, and other electronic devices, always use certified accessories and proper cables. Only

English

the cables specified in the *Philips Axon Instructions for Use* are approved for safe and compliant operation. The use of any other cable is strictly prohibited and may result in interference or device malfunction.

REGULATORY COMPLIANCE

INFORMATION

Philips hereby declares that this device complies with essential requirements and other relevant provisions of the RE Directive 2014/53/EU concerning radio equipment and carries CE-marking accordingly.

For a copy of the full RED Declaration of Conformity, contact your Philips representative.

This equipment may be operated in: AT BE BG HR CY CZ DK EE FI FR DE GR HU IS IE IT LV LI LT LU MT NL NO PL PT RO SK SI ES SE CH TR

Certain countries have specific restrictions for, or prohibitions on devices that operate in the 5 GHz band. Specifically in certain European countries, for example, some frequencies should be restricted to indoor use. You are advised to respect local requirements.

Restriction in Norway: The use of the 2.4 GHz band is prohibited within a 20 km radius around Ny-Ålesund, Svalbard.

EU Authorization Representative and

Importer: Philips Medical Systems

Nederland B.V. - Veenpluis 6 - 5684 PC Best
- The Netherlands

This device complies with essential requirements and other relevant provisions of the 2017 Radio Equipment Regulations, No. 2017 SI 2017/1206. This equipment may be operated in

the United Kingdom. Certain countries have specific restrictions for, or prohibitions on devices that operate in the 5 GHz band.

Specifically in certain European countries, for example, some frequencies should be restricted to indoor use. You are advised to respect local requirements.

For a copy of the full UKCA Declaration of Conformity, contact your Philips representative.

UK Authorization Representative and

Importer:

Philips Electronics UK Limited - Ascent 1 -
Aerospace Boulevard - Farnborough GU14
6XW - United Kingdom

This symbol on the Philips Axon indicates that cables should be collected separately and not disposed of with household waste. As you use Philips Axon, you will accumulate solid wastes that require proper disposal or recycling. These include system components and packaging materials.

Recycling and the Environment

Improper disposal of IT and medical equipment can have a negative impact on health and the environment. We recommend that you dispose of Philips products, including all electronic devices, cables, and so on, at an appropriate facility to enable recovery and recycling. You can also recycle packaging and guides according to your local recycling regulations.

Components

At the end of its service life, the products described in this manual, as well as their accessories, must be disposed of in compliance with the guidelines regulating the disposal of such products. If you have questions concerning disposal of the product, contact your Philips representative.

FCC and ISED (USA Federal Communications Commission and Innovation, Science and Economic Development Canada)

Philips Axon complies with Part 15 of the FCC Rules and ISED Canada license-exempt RSS standard(s). Operation is subject to two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

FCC Note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or

English

television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna. Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device meets the requirements specified in Part 15E, Section 15.407 of the FCC Rules.

Philips is not responsible for any radio or television interference caused by unauthorized changes or modifications to this equipment. Unauthorized changes or modifications could void the user's authority to operate the equipment. Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Shielded cables might be utilized with this unit

to guarantee the compliance with B-Class FCC thresholds. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Info-communications Media Development Authority (IMDA)

Philips Axon is an IMDA registered product.

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All rights are reserved. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical or otherwise, is prohibited without the prior written consent of the copyright owner.
Philips North America LLC