English Arabic Bulgarian Chinese (traditional) Croatian Czech Danish Dutch Estonian Finnish
French German Greek Hungarian Icelandic Italian Japanese Korean Latvian Lithuanian
Maltese Norwegian Polish Portuguese Romanian Slovak Slovenian Spanish Swedish

Neuron 3 Safety Instructions

PHILIPS

Contents

English.....	2
اللغة العربية.....	9
български.....	12
繁體中文.....	18
Hrvatski.....	22
Čeština.....	28
Dansk.....	33
Nederlands.....	38
Eesti.....	44
suomi.....	49
Français.....	54
Deutsch.....	60
Ελληνικά.....	66
Magyar.....	72
Íslenska.....	78
Italiano.....	83
日本語.....	89
한국어.....	93
Latviešu.....	97
Lietuvių k.....	102
Malti.....	107
Norsk.....	112
Polski.....	117
Português.....	123
Română.....	129
Slovenčina.....	135
Slovenščina.....	141
Español.....	146
Svenska.....	152
Philips.....	157

English

IMPORTANT SAFETY INFORMATION

Before using the SmartLinx Neuron 3, make sure to read all the safety information provided below, as well as the *SmartLinx Neuron 3 Installation and Maintenance Guide* available at

customers.capsuletech.com.

Scan the QR code below to access the customer portal and find the relevant product documentation using your product's label.

- Do not modify the SmartLinx Neuron 3, cables, or any part of the Capsule MDIP software, or mount the device in any way other than instructed in Philips documents. Any modification to the hardware or software can prevent proper functioning and cause damage to the connected medical device.
- Be careful to cut only the tape and not, for example, the SmartLinx Neuron.
- Fire and explosion hazard. Do not operate the SmartLinx Neuron in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide; in oxygen-enriched environments; or in any other potentially explosive environment.
- Do not stack the SmartLinx Neuron or place it next to other equipment. If you do, then check it for normal operation running its usual configuration.
- Avoid mounting the Neuron in such a way that prolonged physical contact or pressure is exerted on any part of the Neuron casing or attached components. Such pressure or contact may prevent correct cooling of the device and cause it to overheat.
- Do not place the SmartLinx Neuron inside furniture or inside any enclosed space. This could prevent correct cooling of the device and interfere with radio communications.
- Operate the SmartLinx Neuron 3 only within the environmental conditions described in the "Technical characteristics" of the *Neuron 3 Installation and Maintenance Guide*.
- Secure overhead installations of SmartLinx Neurons to prevent falls.

Perform regular maintenance checks to ensure stability.

- To avoid the risk of electric shock, this equipment must only be connected to an electrical outlet with protective earth.
- Always use the power supply unit provided with the equipment: SL-NU3-PS (Model: 989907000055)
- Do not use extension cords or adapters of any type.
- The power cable, supply, and plug must remain intact and undamaged.
- The power plug must remain accessible to unplug the device from the electrical outlet if necessary.
- For safety, cables are designed to prevent inadvertent disconnection if someone accidentally pulls on them. However, make sure that you route cables so that they do not present a tripping hazard.
- Avoid connecting a passive Power-over-Ethernet (PoE) network cable to a serial port as PoE current can damage the device.
- Use the rear-accessible USB ports to connect only non-medical devices to the SmartLinx Neuron. An exception to this warning involves USB-

connected medical devices that are isolated; it is possible to connect such devices to the rear-accessible USB ports.

- Avoid holding down the ON/OFF button. After four seconds, the resulting hard shutdown can have a negative impact on device stability.
- Defective batteries can damage the SmartLinx Neuron. If the battery shows any signs of damage or cracking, it must be replaced immediately and only with a battery approved by Philips.
- Improper disposal of batteries may create an explosion or contamination hazard. Never dispose of batteries in refuse containers. Always recycle batteries according to local regulations. Never dispose of batteries in refuse containers. Do not wrap them in metal or aluminum foil. Wrap them in newspaper before disposing of them. Do not burn them. Battery may explode if overheated.
- To avoid the possibility of an explosion, use only batteries supplied by Philips or a Philips-approved reseller. If you use any other battery, you risk an explosion.

English

- Before you remove a battery from a SmartLinx Neuron which is powered on, make sure that it continues to receive power, either from the electrical outlet, or from at least one charged battery that is installed in the SmartLinx Neuron.
- Electric shock hazard. Do not open the SmartLinx Neuron or attempt repairs. The SmartLinx Neuron has no user-serviceable internal parts. Only perform routine cleaning and maintenance procedures specifically described in this manual. Inspection and servicing of internal parts must only be performed by qualified service personnel
- Since the SmartLinx Neuron is not completely watertight:
 - Do not spray cleaning liquid or foam on the SmartLinx Neuron or submerge or soak it for cleaning.
 - Never spray or squirt any type of liquid directly on the SmartLinx Neuron; spray the liquid, gel, or foam on a cloth and then rub that cloth on the device.
 - Do not spill liquids on or around the unit.
 - Do not expose the power cable or power brick to liquid.
 - Make sure that the cover on the USB port is securely fastened.
 - Do not use any other agent than those listed in the Smartlinx Neuron 3 Installation and Maintenance Guide (Contact Philips for assistance if your cleaning agent is not on the list.)
 - Do not use harsh solvents.
 - Do not clean, disinfect, or sterilize any part of the system by autoclaving or with the use of ethylene oxide gas (gas sterilization).
 - Do not touch, press, or rub the display panel with abrasive cleaning compounds, instruments, brushes or rough-surface materials.
 - Power off the SmartLinx Neuron before you inspect the power cable and power brick.

English

Electrical specifications

External Power Supply: 100–240 V AC,
50/60 Hz, Power Consumption: 65W max
Electrical Ratings: 20 V DC, 3.25 A max
Internal Battery: Lithium-Ion Battery Pack
3S1P 3020 mAh

Frequency bands and power of the Smartlinx Neuron 3

Wi-Fi 2.4 GHz band: 2412 MHz to 2472 MHz
(Power < 19.9dBm)

Wi-Fi 5.0 GHz band: 5180 MHz to 5240 MHz
, 5260 MHz to 5320 MHz, 5500 MHz to
5700 MHz, 5725 MHz to 5850 MHz (Power
< 22.92 dBm)

Operating environment

Temperature: 0°C to +40°C (+32°F to
+104°F)

Humidity: 15% to 95% RH non-condensing
Atmospheric pressure: 700 hPa to 1060 hPa

Transport and storage environment

Temperature: -20°C to +60°C (-4°F to
+140°F)

Humidity: 15% to 95% RH non-condensing
Atmospheric pressure: 500 hPa to 1060 hPa

Important note

Any changes or modifications to this
product are strictly forbidden. Such actions
will immediately void all safety and wireless
communication certifications and revoke
your authorization to operate the device.

This product has been evaluated and
certified to meet safety and wireless
standards when used with approved
accessories and shielded cables that comply
with regulatory requirements. To prevent
interference with radios, televisions, and
other electronic devices, always use
certified accessories and proper cables. Only
the cables specified in the *Neuron 3
Installation and Maintenance Guide* are
approved for safe and compliant operation.
The use of any other cable is strictly
prohibited and may result in interference or
device malfunction.

REGULATORY COMPLIANCE INFORMATION

Philips hereby declares that this device
complies with essential requirements and
other relevant provisions of the RE Directive
2014/53/EU concerning radio equipment
and carries CE-marking accordingly.
For a copy of the full RED Declaration of
Conformity, contact your Philips
representative.

This equipment may be operated in: AT BE
BG HR CY CZ DK EE FI FR DE GR HU IS
IE IT LV LI LT LU MT NL NO PL PT RO
SK SI ES SE CH TR

Certain countries have specific restrictions

English

for, or prohibitions on devices that operate in the 5 GHz band. Specifically in certain European countries, for example, some frequencies should be restricted to indoor use. You are advised to respect local requirements.

Restriction in Norway: The use of the 2.4 GHz band is prohibited within a 20 km radius around Ny-Ålesund, Svalbard.

EU authorization representative and importer

Philips Medical Systems Nederland B.V.

Veenpluis 6 - 5684 PC Best

The Netherlands

This device complies with essential requirements and other relevant provisions of the 2017 Radio Equipment Regulations, No. 2017 SI 2017/1206. This equipment may be operated in the United Kingdom. Certain countries have specific restrictions for, or prohibitions on devices that operate in the 5 GHz band. Specifically in certain European countries, for example, some frequencies should be restricted to indoor use. You are advised to respect local requirements. For a copy of the full UKCA Declaration of Conformity, contact your Philips representative.

UK authorization representative and importer

Philips Electronics UK Limited

Ascent 1

Aerospace Boulevard

Farnborough GU14 6XW

United Kingdom

This symbol on the SmartLinx Neuron 3 indicates that batteries and cables should be collected separately and not disposed of with household waste.

As you use the SmartLinx Neuron 3, you will accumulate solid wastes that require proper disposal or recycling. These include system components, batteries, and packaging materials.

Recycling and the environment

Improper disposal of IT and medical equipment can have a negative impact on health and the environment. We recommend that you dispose of Philips products, including all electronic devices, cables, batteries, and so on, at an appropriate facility to enable recovery and recycling. You can also recycle packaging and guides according to your local recycling regulations.

Components

English

At the end of its service life, the products described in this manual, as well as their accessories, must be disposed of in compliance with the guidelines regulating the disposal of such products. If you have questions concerning disposal of the product, contact your Philips representative.

Batteries

The sealed, rechargeable batteries contain lead and can be recycled. Discharge (deplete) batteries prior to disposal. Place the battery in packaging which electrically isolates its contents. Do not puncture or place the battery in a trash compactor. Do not incinerate the battery or expose it to fire or high temperatures. Dispose in accordance with regional body-controlled guidelines and hospital protocol.

FCC and ISED

SmartLinx Neuron complies with Part 15 of the FCC Rules and ISED Canada licence-exempt RSS standard(s). Operation is subject to two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

FCC note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

Reorient or relocate the receiving antenna.

Increase the separation between the equipment and receiver.

Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

Consult the dealer or an experienced radio/TV technician for help

This device meets the requirements

English

specified in Part 15E, Section 15.407 of the
FCC rules.

Philips is not responsible for any radio or television interference caused by unauthorized changes or modifications to this equipment. Unauthorized changes or modifications could void the user's authority to operate the equipment. Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment. Shielded cables might be utilized with this unit to guarantee the compliance with B-Class FCC thresholds. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

**Info-communications Media Development
Authority (IMDA)**

The SmartLinx Neuron is an IMDA registered product.

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All rights are reserved. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical or otherwise, is prohibited without the prior written consent of the copyright owner.
Philips North America LLC